

Incidin™ Wipes FlexPack



Dry Wipes System

Hygienic disposable wipe dispenser for surface disinfection. To combine with one of the validated cleaning and disinfection solutions.

In combination with validated surface disinfectants, Incidin™ Wipes FlexPack is the advanced solution for easy and efficient surface cleaning and disinfection.

The advantages of Incidin™ Wipes FlexPack

- EASY - ready-to-use in 30 minutes
- STABLE - down to the last wipe
- EFFICIENT - saves time, space, costs and waste

Incidin™ Wipes FlexPack meets the requirements of German health authority RKI (Robert Koch Institute) and VAH (Association of Applied Hygiene) recommendation for risk areas in hospitals^{1,2}



Incidin™ Wipes FlexPack

Hygienic disposable wipe dispenser for surface disinfection.

To combine with one of the validated cleaning and disinfection solutions.


Product characteristics

- roll of 99 wipes made of high-quality dry and lint-free non-woven PET material
- wipe size 20 x 38 cm
- in-use stability after impregnation with one of the validated surface disinfectants: 31 days

Instructions for use



Wear gloves.

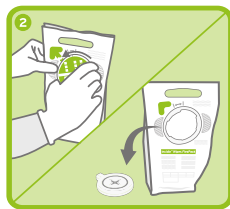


Hold Incidin™ Wipes FlexPack on marked wings to the left or right of the lid. Break the seal by unscrewing the lid, turning it counter-clockwise. Place the lid with the top on the work space.




Pull-out first wipe partially.




Pour in 2.5 L of one of validated disinfection solutions (4a concentrate, 4b ready to use; use a previously disinfected measuring container). Only use disinfectants validated to be combined with Incidin™ Wipes FlexPack.

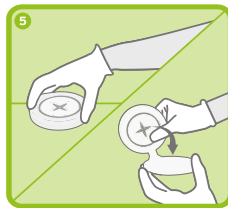


Open the lid.

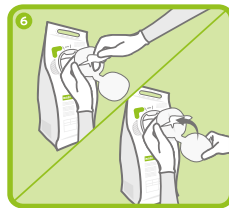


Pull the first wipe through and close the lid.

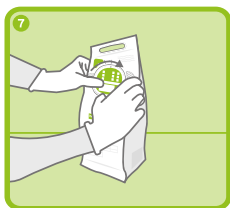

Hold Incidin™ Wipes FlexPack on marked wings to the left or right of the lid. Screw the lid onto the bag turning it clockwise, starting at the mark until it snaps.

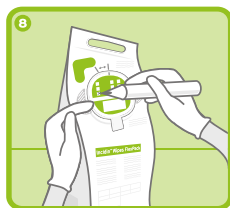

Mark the lid label with the required information. After filling wait appr. 30 min. to have wipe roll impregnated with soaking solution.

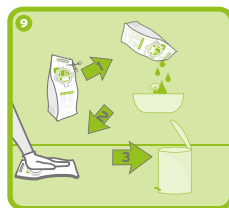

In case of any liquid residues when discarding, cut corner to dispose of this easily and discard Incidin™ Wipes FlexPack.

Packaging and ordering information

Product	Packaging unit	Order code
Incidin™ Wipes FlexPack with green lid	6 x 99 wipes / case	XXXXX
Incidin™ Wipes FlexPack with red lid	6 x 99 wipes / case	XXXXX
Incidin™ Wipes FlexPack with blue lid	6 x 99 wipes / case	XXXXX

ECOLAB EUROPE

Hofwiesenstrasse 349
CH-8050 Zürich
Switzerland

www.ecolab.com

ECOLAB DEUTSCHLAND GmbH

Ecolab-Allee 1
40789 Monheim am Rhein
Germany

Application area

Hygienic disposable wipe dispenser in the form of a stable stand-up pouch with carrying handle. Transparent windows allow for checking the consumption.

Each dispenser contains a roll of 99 wipes made of high-quality dry and lint-free non-woven PET material (wipe size 20x 38 cm) to combine with the validated surface disinfectants.

Product features

EASY - ready-to-use in 30 minutes

- The wipe dispenser is easy to fill and to mark.
- It is ready-to-use in 30 minutes.
- The carrying handle facilitates transport to the POC (Point of Care).
- The colour coding allows for filling with different products or use in specific areas / situations.

STABLE - down to the last wipe

- The stand-up pouch stands firmly on e. g. the cleaning trolley until the last wipe is removed.

EFFICIENT - saves time, space, costs and waste

- Saves time by making transport and processing unnecessary.
- Saves storage space.
- Saves costs through more efficient processes.
- Saves waste by reducing waste volume.

COMPATIBLE

Incidin™ Wipes FlexPack can be combined with the following products:

- Incidin™ Pro
- Surfanios Premium
- Incidin™ OxyFoam S
- Incidin™ OxyFoam NG
- Surfa'Safe Premium
- Incidin™ Liquid
- Incidin™ Premium Foam

References

¹ Commission for Hospital Hygiene and Infection Prevention (KRINKO). Hygiene requirements for cleaning and disinfection of surfaces: recommendation of the Commission for Hospital Hygiene and Infection Prevention (KRINKO) at the Robert Koch Institute. GMS Hyg Infect Control. 2024 Mar 5;19:Doc13. doi: 10.3205/dgkh000468. PMID: 38655122; PMCID: PMC11035912.

² Mitteilung der Desinfektionsmittel-Kommission im VAH unter Mitwirkung der „4+4-Arbeitsgruppe“, Zur Verwendung von Tuchspendensystemen in Bereichen mit besonderem Infektionsrisiko, HygMed 2014;39(9):358-359.



FOR PROFESSIONAL USE ONLY.

ECOLAB[®]
PROTECTING WHAT'S VITAL™